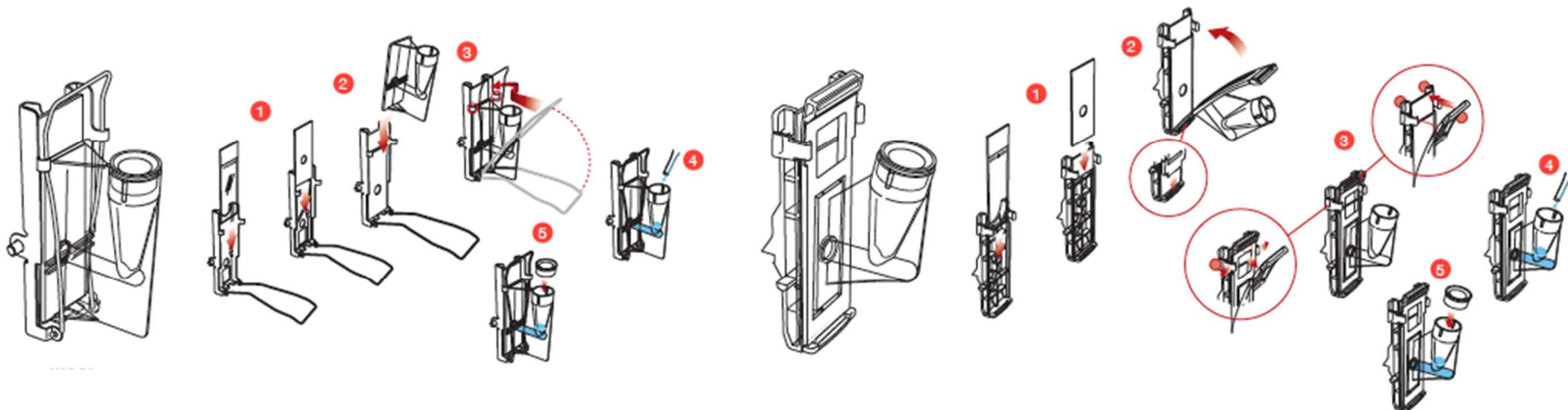
CytoSep™ Instructions for use

EN

INTENDED USE

CytoSep™ cytotag funnels are designed to hold Biological fluids for centrifugation, to allow cells to be concentrated onto a microscope slide for the preparation of samples and further analysis. It is a non-automated, single use device for in vitro diagnostics. **Not automated, not a qualitative, semi-quantitative or quantitative device. Used by laboratory professionals only.**

INSTRUCTIONS FOR USE

1. Insert the slide inside the plastic or metal clip, making sure the slide's label frosted side is on top and facing the funnel
2.
 - a) Place the appropriate filter over the slide
 - b) Fit the matching funnel on top of the filter
 - c) For the pre-assembled funnel with filter, place the unit over the slide
3. Secure the funnel chamber, filter and slide using the metal or plastic clip
4. Fill the funnel with the sample
5. Place the cap on the funnel to prevent spillage and avoid aerosols
6. Load the funnel into the centrifuge by following the equipment operator's guide

Product Description:

- White filter card for sample volumes of up to 0.5 ml
- Brown filter card for slower absorption of smaller volumes up to 0.4 ml, ideal for cerebrospinal fluid
- M964-10 and M965-10 single funnels are for samples up to 0.5 ml
- M964-20 and M965-20 double funnels are for 2 separate samples that are run simultaneously
- M964B* metal clips securely hold the single or double funnel and their components
- M965B plastic clip securely hold the single or double funnel and their components
- The M965FW white filter, M965FWD white double filter, M965FT tan filter and M965C caps are compatible with M964 and M965 funnels
- The M965-40 Mega funnel with plastic clip do not require use of any filters (follow laboratory procedures). They are for sample volumes of up to 6 ml
- The M965FWDV white filter paper is compatible with the Shandon™ TPX Single sample chamber

*M964B is compatible with the Hettich Rotofix 32A

*Shandon and Cytospin are registered trademarks of Thermo Fisher Scientific Inc.

Storage conditions: No special storage condition required

Warning: Safely dispose of infectious substances following applicable rules and regulations on waste disposals. Please report any serious incident to Simport and to the competent authority of the Member State where you are established.

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

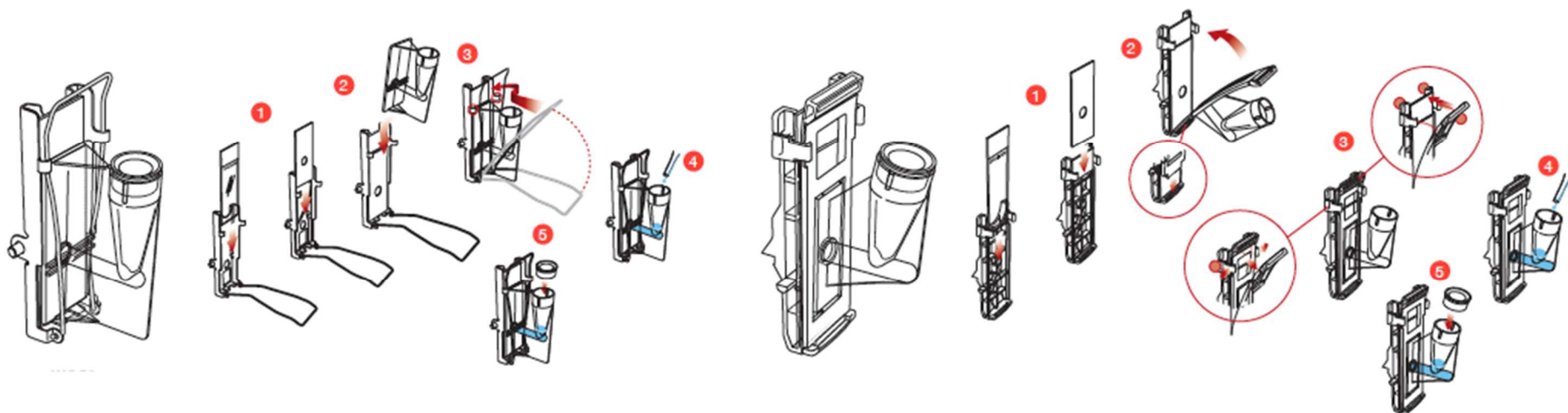
ETI-M964 IFU REV05. Date d'émission / Issue Date: 2022-06-23 (revisions highlighted in yellow and warnings added)

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EC REP EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

CH REP MedEnvoy Switzerland
Gothardstrasse 28
6302 Zug
Switzerland

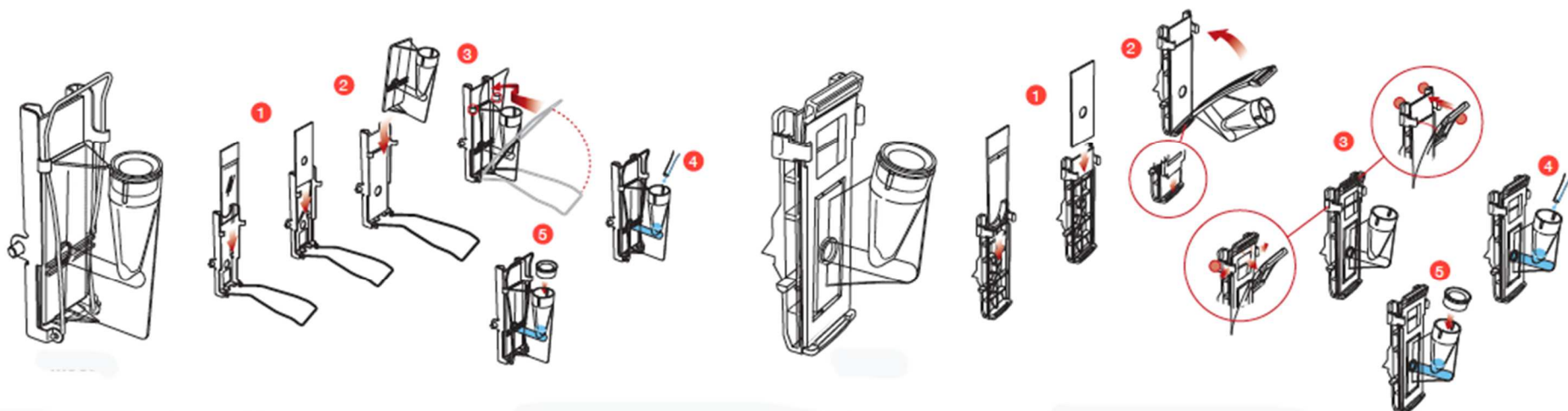
ETI-M964 IFU REV05. Date d'émission / Issue Date: 2022-06-23 (revisions highlighted in yellow and warnings added)

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EC REP
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

CH REP
MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

IVD **CE**

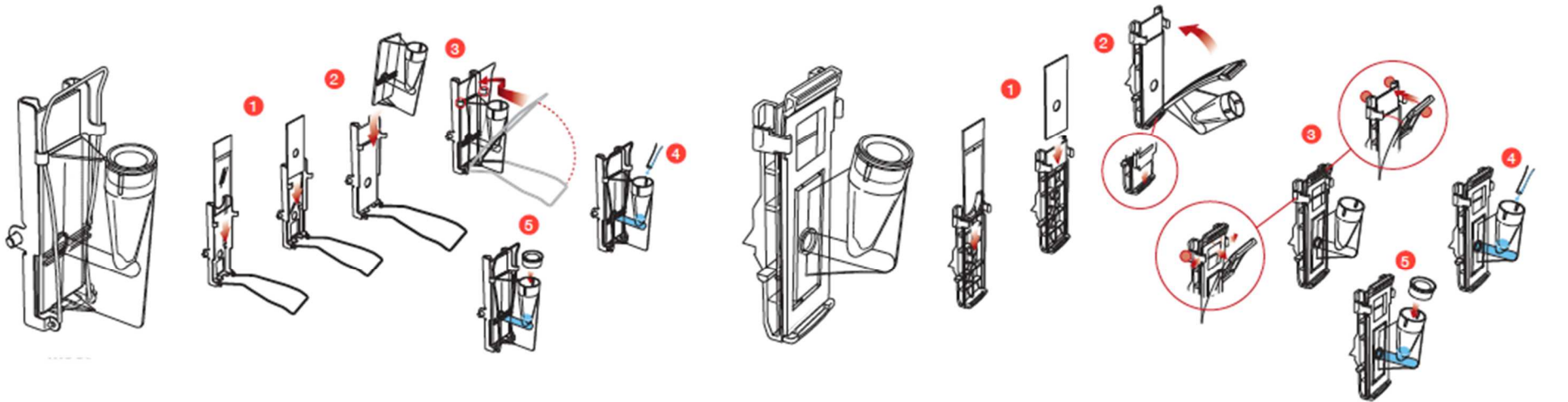
ETI-M964 IFU REV05. Date d'émission / Issue Date: 2022-06-23 (revisions highlighted in yellow and warnings added)

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Simport® Scientific Inc., 2588, Bernard-Pilon, Beloeil (Québec) J3G 4S5 Canada